

Scan Bodies, Digital Libraries and Titanium Bases



**Guidelines to Download and Install the
PREAT Library Files for ExoCad Titanium
Bases**

Step 1. Download files for ExoCad at
www.preat.com/portfolio/digital-libraries/




ExoCad


Digital Library Download for ExoCad ▶

ExoCad Guideline to Download and Install ▶


Step 2. Save file to a location on the computer that is easily accessible.

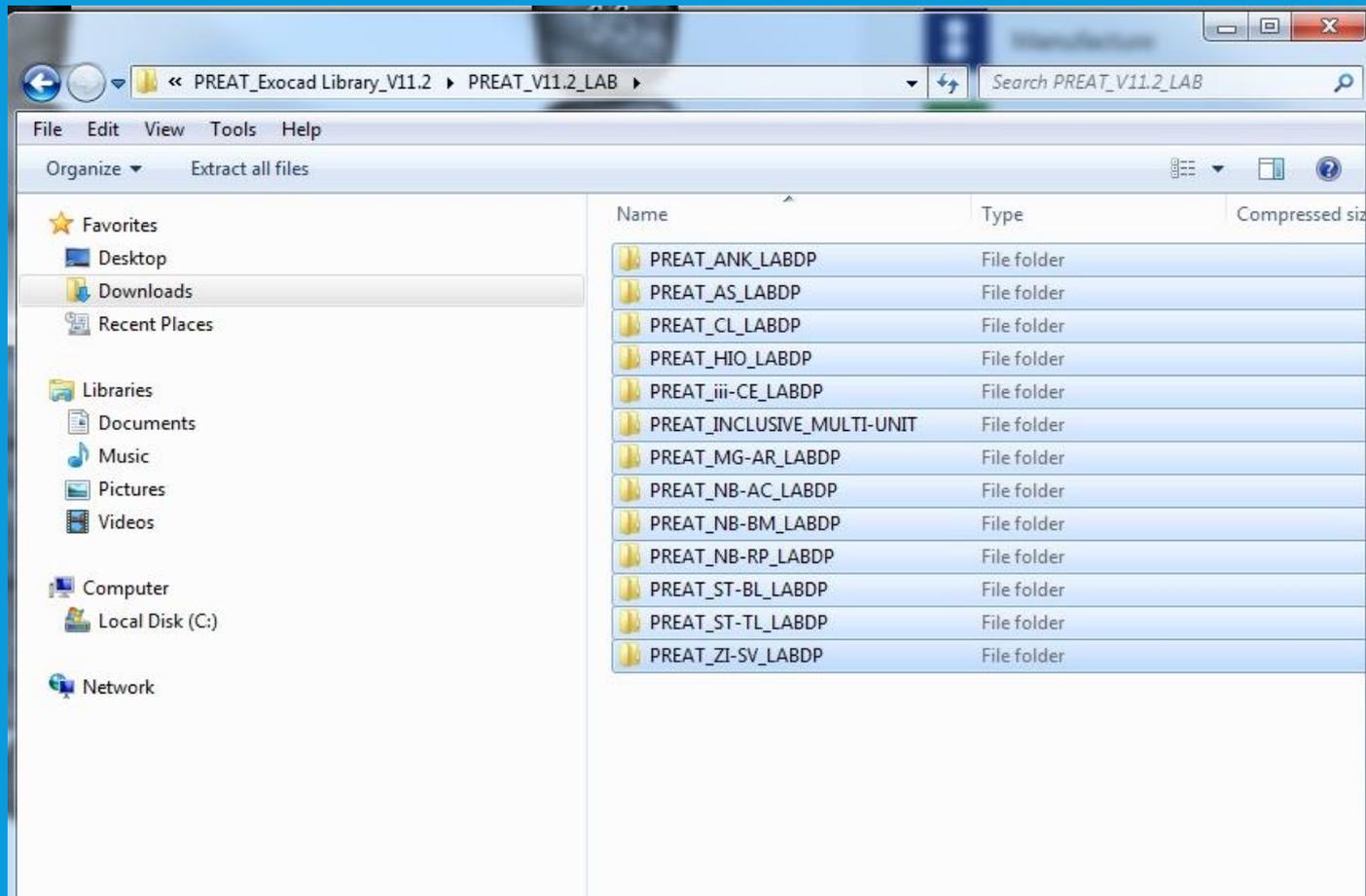


Step 3. Open Windows Explorer. Navigate to and click on the Dental CAD Software folder.

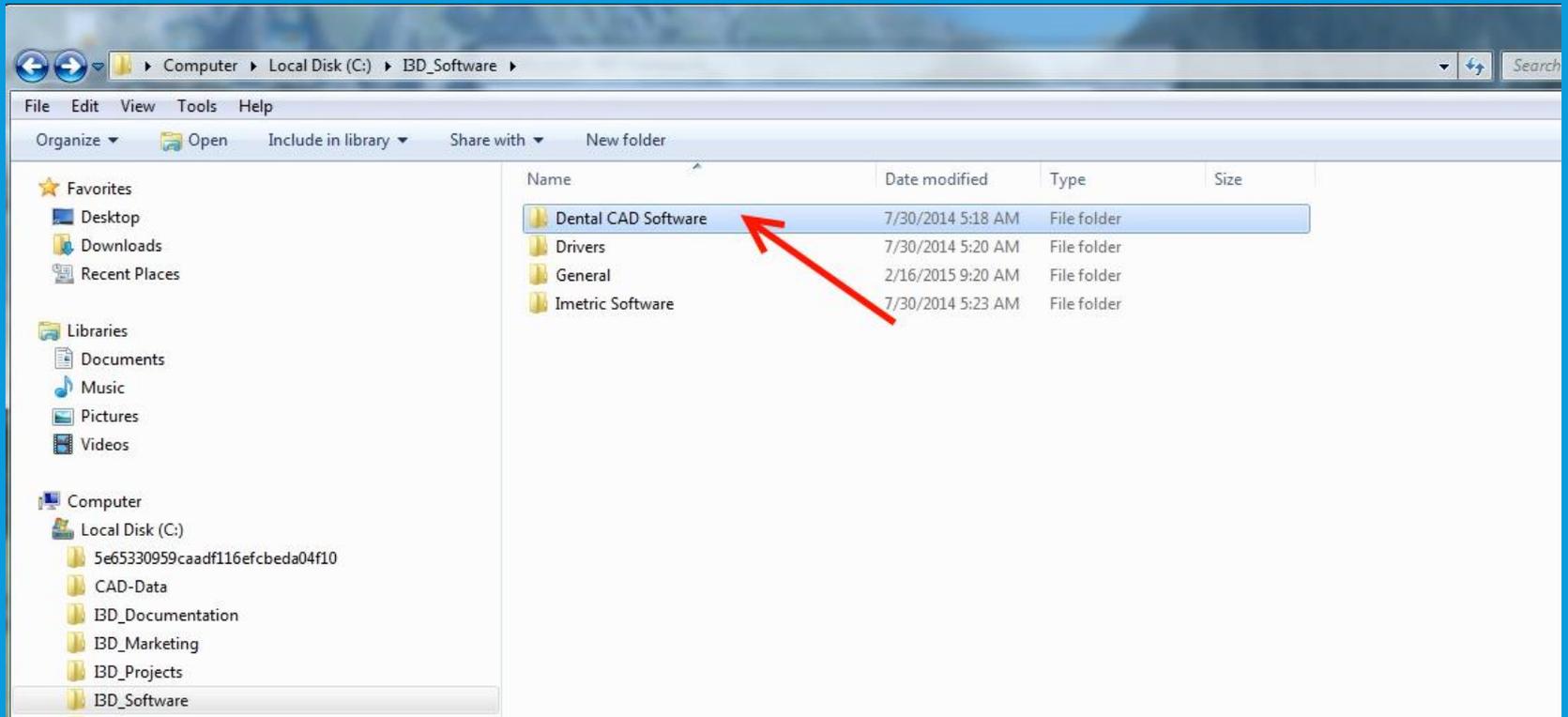


Step 4. Click on the ExoCad Folder.

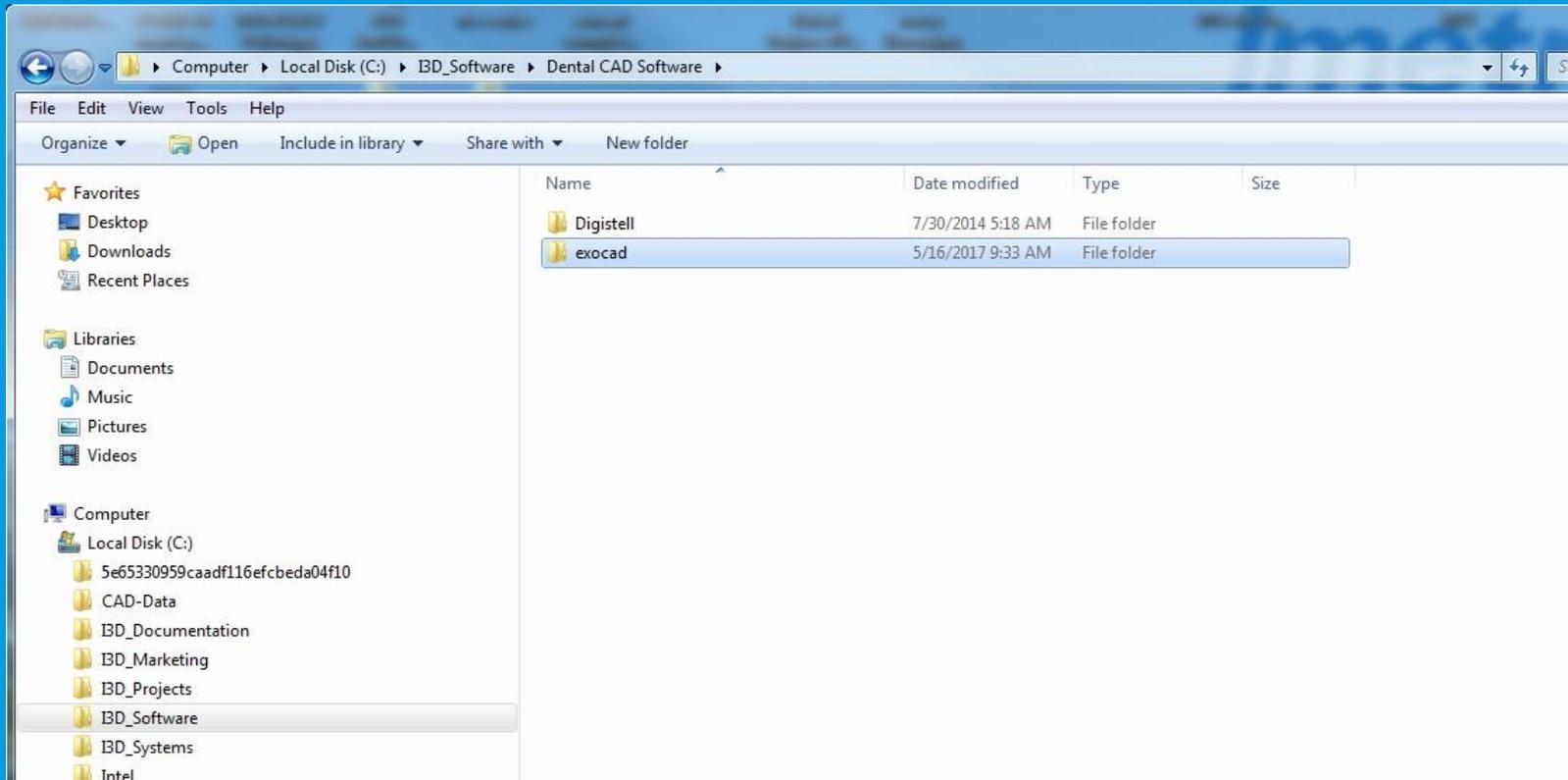


5. Click on DentalCADApp folder.

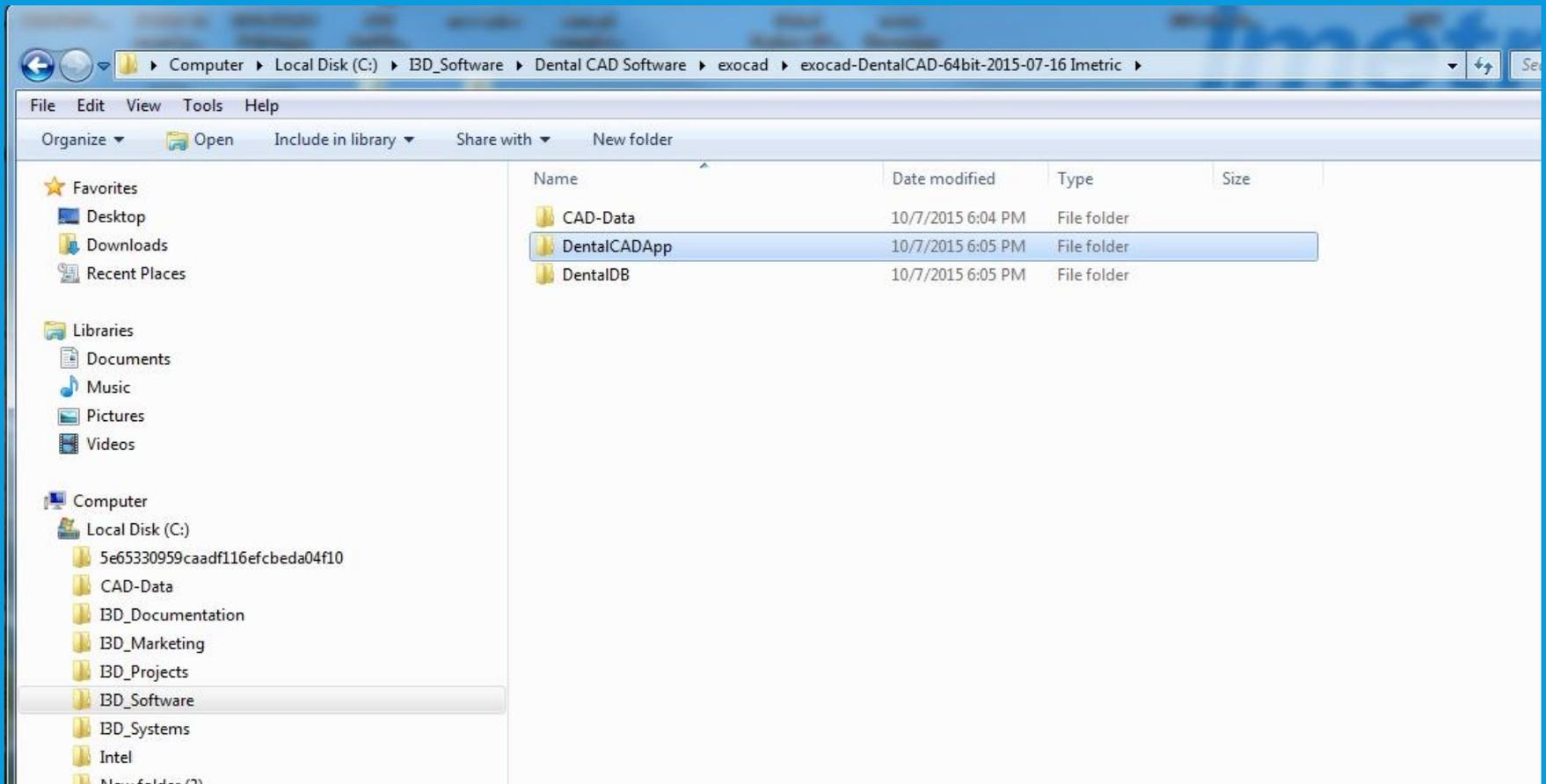


Step 6. Click on the Library folder.

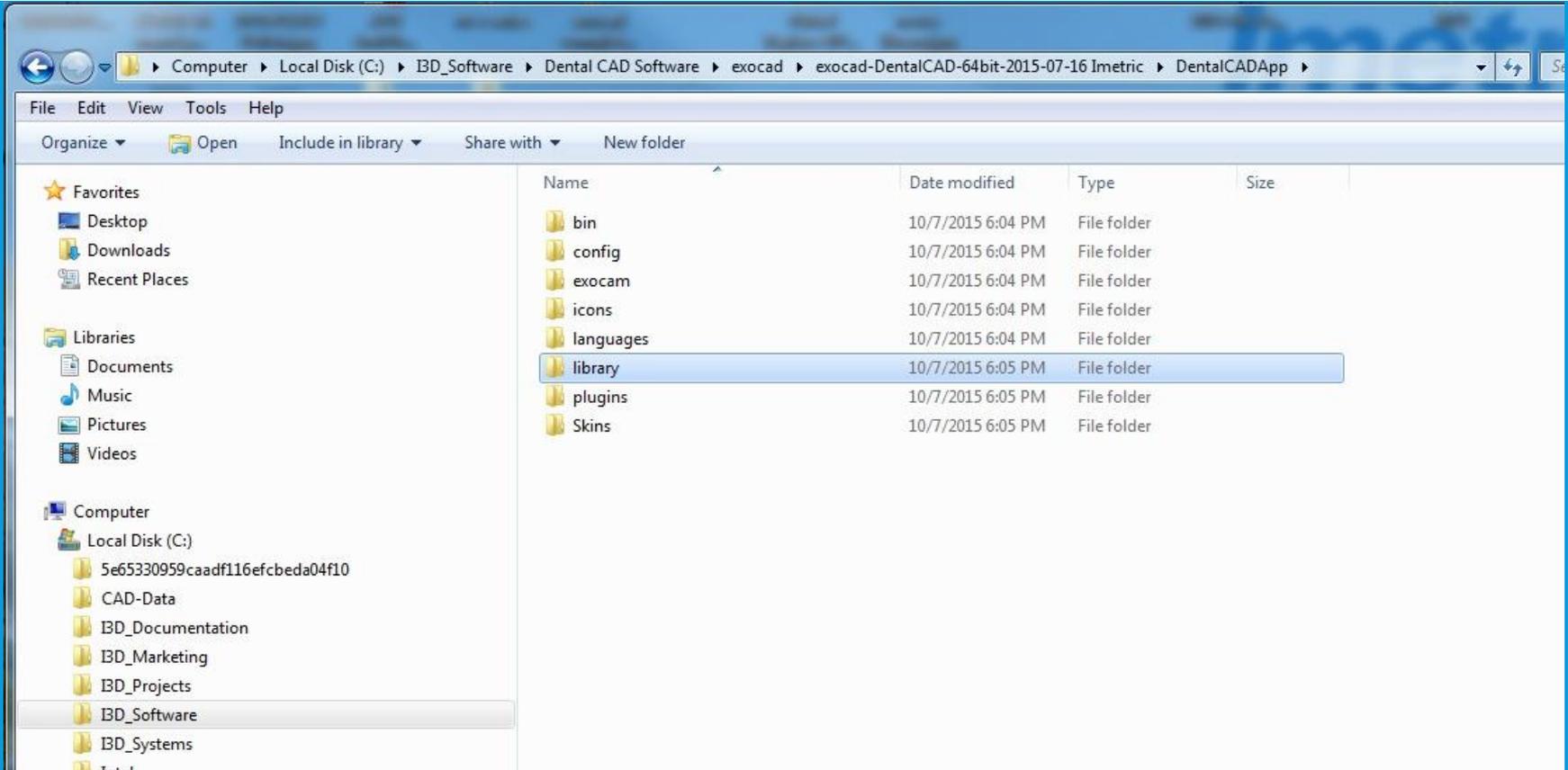


Step 7. Click on the Implant folder.

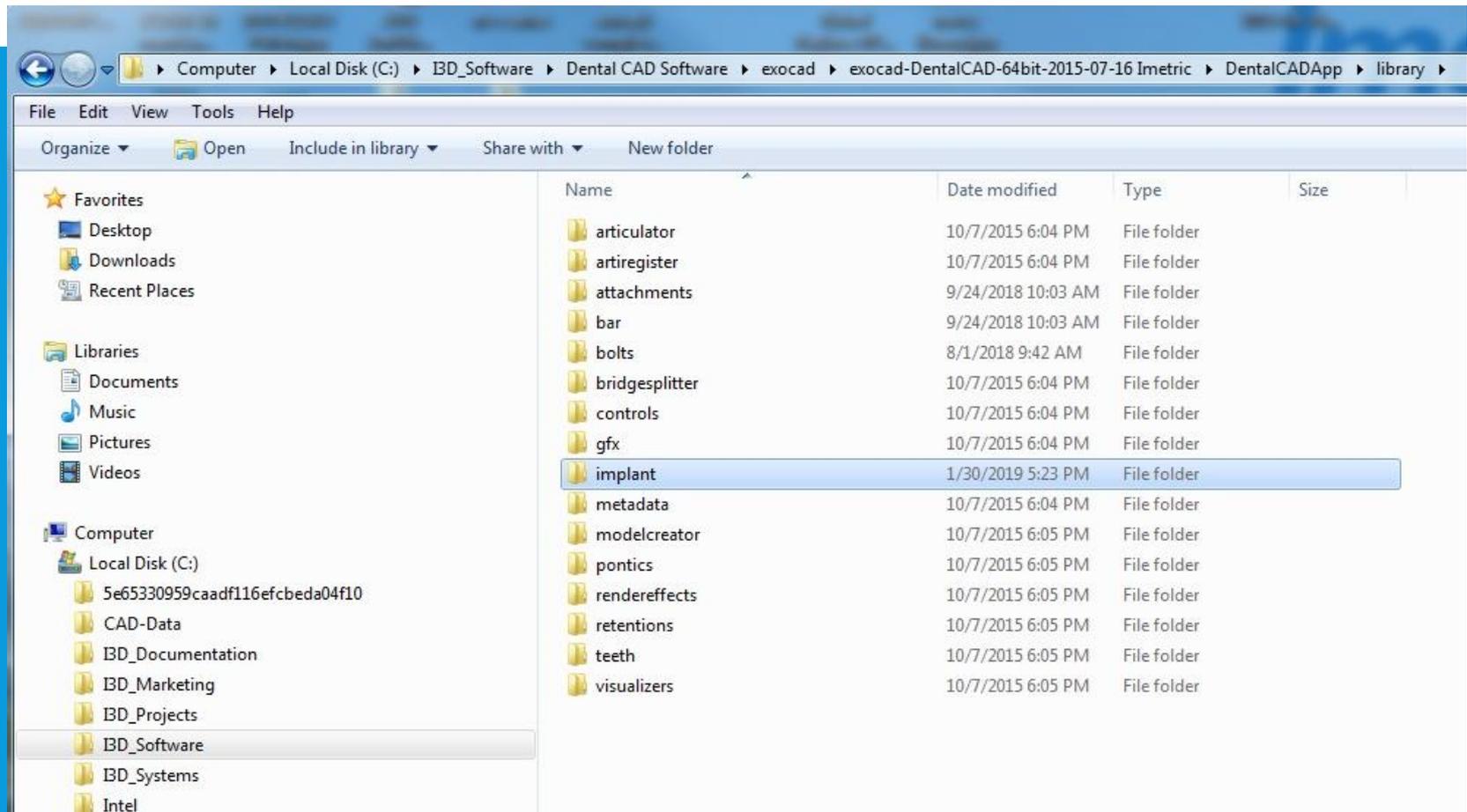

Step 8. Copy all the implant library folders from the downloaded location.

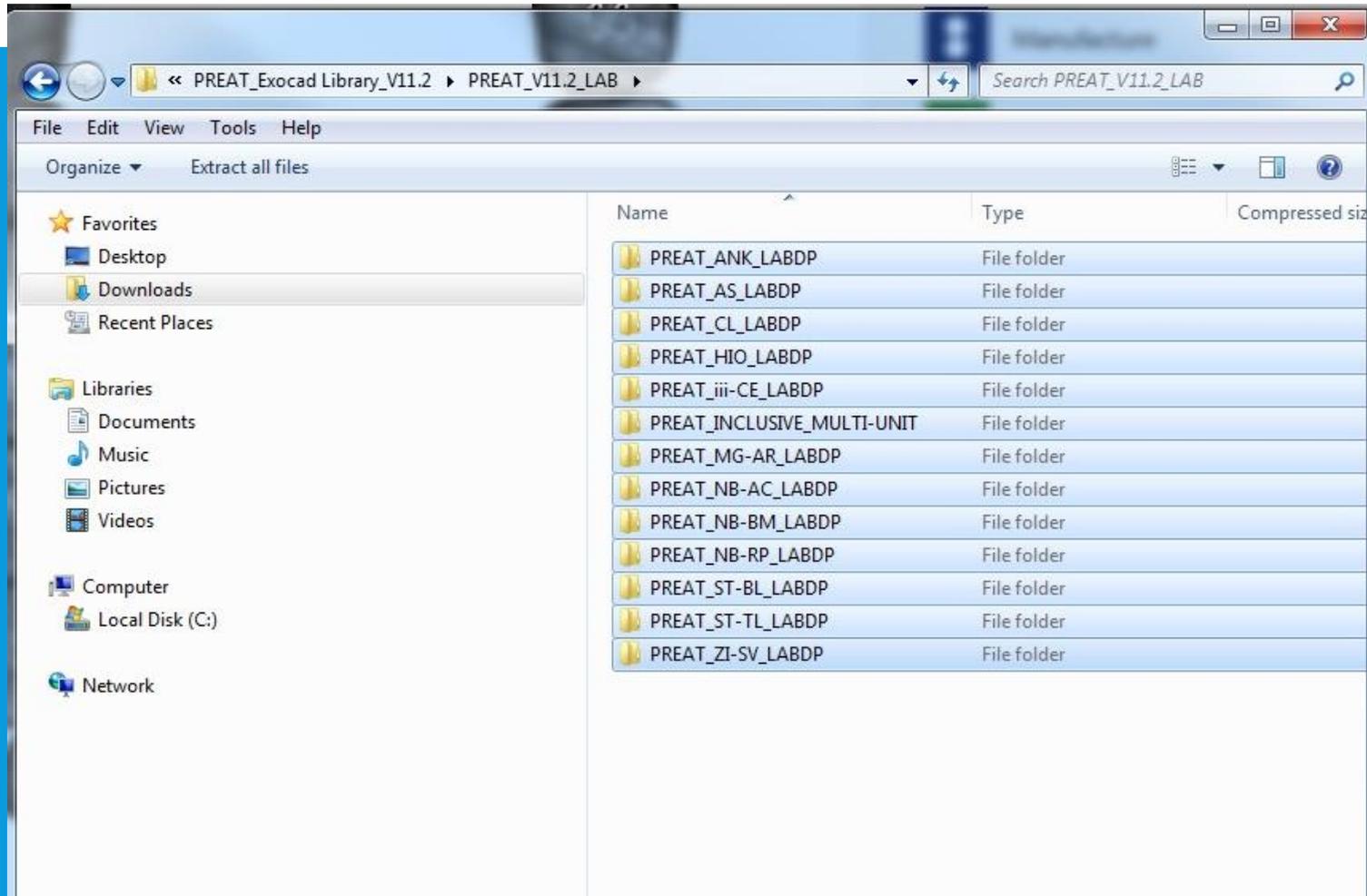

Step 9. Paste the folders in the Implant library folder.

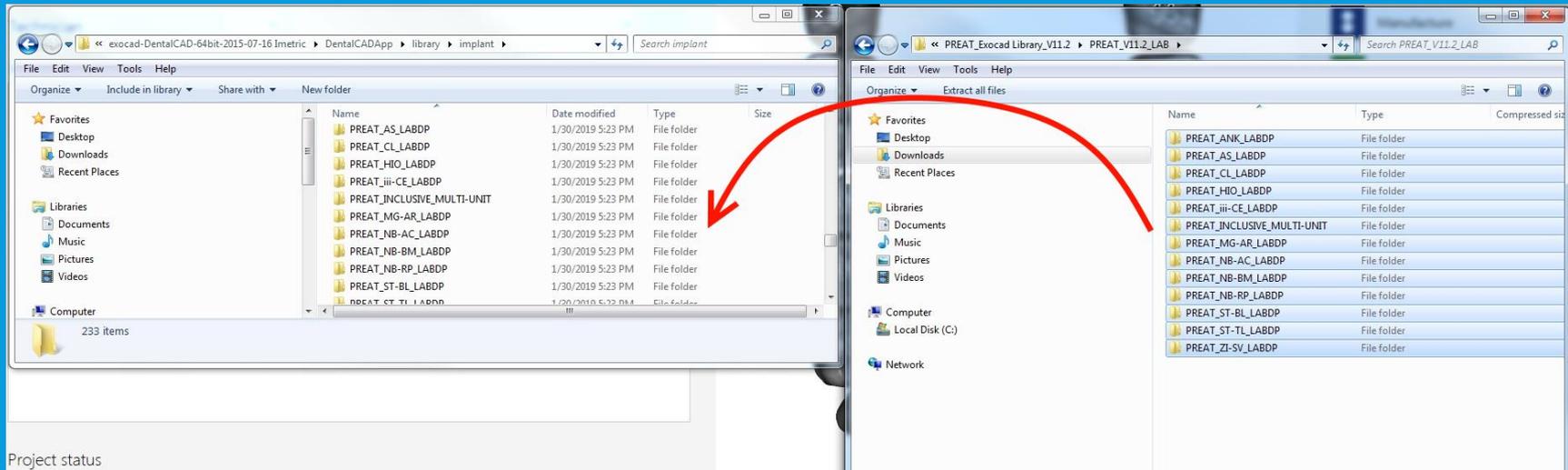

Project status

Thank You



**If you still require technical assistance, please
contact PREAT at:**

1-800-232-7732 or technical@preat.com