



Bidra Conversion Smart Polishing Cap



Figure 1

1. Screw the Bidra Conversion Smart Polishing Cap into the metal coping
2. Begin the conversion process by adding resin (See Figure 1)



Figure 2

3. Allow acrylic resin to completely set (See Figure 2)
4. Watch the Bidra Conversion Smart Polishing Cap in action!



Figure 3

5. If applicable: after anterior acrylic has set, remove and screw Bidra Conversion Smart Polishing Caps onto posterior copings
6. Repeat Steps 2-4 (See Figure 3)